# UK: CANNASEN® CBD Pain Patch Instructions For Use

Please read instructions for use carefully before using the CANNASEN® CBD Pain Patch.

**How does CANNASEN® CBD Pain Patch work?**

CANNASEN® CBD Pain Patch is a topical product with acrylate copolymer solution, which:

- Helps relieve muscle/joint pain; and
- Produces a conductive cooling effect

CANNASEN® CBD Pain Patch is a medical device. Efficacy and safety have been tested according to the European Directive for Medical Devices (93/42/EEC).

**Active Principle of CANNASEN® CBD Pain Patch**

CANNASEN® CBD Pain Patch is used for the relief of muscle/joint pain (such as arthritis, back pain, sprains, sports injuries). CANNASEN® CBD Pain Patch has cooling and pain-relieving properties. The cooling effect, which is perceived as pleasant, provides a long lasting pain relief.

**Application areas**

For cooling muscles and joints

For relief of pain and swelling (such as arthritis, back pain, sprains, sports injuries).

**Dosage instructions, type and duration of application**

CANNASEN® CBD Pain Patch should only be used on the skin. Clean and dry the affected area before applying the product on the skin. Remove the cover from the patch and apply it to the affected area. Do not apply the patch to injured or irritated skin (e.g. scratched, sunburned, infected or on skin with cuts or rash). The patch can be used once a day. Follow all instructions on the product packaging.

Remove the patch before showering and do not apply it immediately after shower. Do not apply bandages or do not wrap the area where the patch is applied. Do not cover the treated skin area with a heating pad as this may increase the risk for side effects.

If other products are applied to the skin area of the patch - wash it well off and wait around 2 hours before using the patch. Wash your hands well after handling the patch. Avoid getting the patch in your eyes, nose or mouth.

After the patch removal, fold the sticky sides of the used patch and dispose of it in the household waste.

The product is for temporary use.

If there is no improvement 2 - 3 days after starting the treatment or if the symptoms have worsened considerably, please consult a doctor.

**General precautions**

-For external use only.

-Do not use this product on damaged or injured skin.

-Avoid eye contact. Wash hands carefully after use.

-Not recommended for children and adolescents under 14 years of age. Use only after consulting your doctor.

-Keep out of reach of small children.

-Keep out of reach of pets.

-Persons with an allergy to alcohol, solvents or any of the ingredients should not use the product.

-During pregnancy or breastfeeding, use only after consulting a doctor.

**Contraindications**

CANNASEN® CBD Pain Patch must not be used in cases of known hypersensitivity to any of the ingredients.

**Interactions**

To avoid drug interactions, you should apply medication 2 hours before or after use or consult your doctor.

**Incident reporting**

If this product is associated with a serious incident (death of a patient, user or other person; temporary or permanent serious deterioration of the health status of a patient, user or other person; serious risk to public health), this incident must be reported to the manufacturer and the National competent authorities.

**Instructions and information on the shelf life of the medical device**

CANNASEN® CBD Pain Patch may only be used until the date marked "usable until" on the folding box. Due to the nature of the ingredients, a slight color change may occur after some time using the product. This does not affect the quality of the product.

**Dosage form and content**

Original packaging with 8 Patches.

**Composition**

Active ingredients: Acrylate Copolymer, Ethyl Acetate, Ethanol, Heptane, Pentanedione. Other ingredients: Cannabidiol(CBD), Arnica Tincture, Menthol. Contains alcohol. Without preservatives and colourings.

**Storage**

Keep cool (between 10°C and 25°C), dry and protected from sunlight.

**Disposal**

The product can be disposed of with normal household waste.

	 Manufacturer: CanNordic a/s Fruebjergvej 3 DK-2100 Copenhagen	 Symbol CE	 10°C / 25°C Upper / lower temperature limit	 Use by	 Please observe the instructions for use	 Batch number	 Protect from sunlight	 Do not use if the packaging is damaged	 Store in a dry place	 Do not re-use
--	--	---------------	--	------------	---	------------------	---------------------------	--	--------------------------	-------------------

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