Arthritis Gel.

Hvordan virker Arthritis Gel?

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Opbevar tør

Please read this leaflet carefully before using Arthritis Gel.

UK: Arthritis Gel instructions for use

How does Arthritis gel work?

Arthritis Gel is a gel preparation with an alcohol / water mixture, which

- reduces the painful conditions that occur in rheumatoid arthritis
- causes a conductive cooling effect

Arthritis gel is a medical device. Efficacy and safety have been tested according to the European Directive on

Medical Devices (93/42 / EEC).

Active principle of Arthritis gel

Arthritis gel is applied externally in the form of a gel preparation in painful conditions of hot and swollen joints. It is used to support the external treatment of arthritis and reduce the painful conditions that occur in these patients. The main physical effect is caused by the alcohol/water mixture. When the gel is applied on the skin, the alcohol evaporates, causing a faster conductive cooling effect on the treated skin surface.

Application areas

The medical device is used to support the external treatment of arthritis and to reduce the painful conditions of the hot and swollen joints.

Dosing instructions, type and duration of application

When, how often and how long should you use Arthritis Gel?

Depending on the requirements, it can be used several times a day from the tube, applied and massaged to the relevant body area. Recommendation of dosage: apply 0,25 g size of a pea to hand, wrists or elbows, apply 0,5 g size of 2 peas to foot, ankle or knee. If no improvement is seen within 2 - 3 days after the treatment start or if the symptoms have worsened considerably, please consult a doctor.

The product is for temporary use.

People with an allergy to alcohol or an ingredient should not use the product.

Please note

Keep out of reach of children.

There is no evidence against the gel use during pregnancy or lactation.

Contraindications

Arthritis Gel should not be used in case of known hypersensitivity to any of the ingredients. Please do not use the product on the skin surface at the same time as others.

Interactions

To avoid drug interactions, you should apply medications one to two hours apart or consult your doctor.

Instructions and information on the durability of the medical device

Arthritis Gel may only be used up to the date marked " use by" on the carton.

Pharmaceutical form and content

Original packaging with 50 ml gel.

Composition

Active ingredient: alcohol / water mixture

Other ingredients:

Menthol, Sodium Hydroxide Solution, cannabidiol, Acrylate Crosspolymer, Isopropyl Myristate, Camphor, Phytic acid

Without preservatives.

Keep cool, dry and protected from sunlight. Small color changes have no influence on the efficacy of this product. Open tubes have to be used in 30 days.

Meaning of the symbols on the pack:

CannaSen CBD

Symbol manufacturer:
Manufacturer:
Galaxa Pharma ApS
Korskildelund
DK - 2670 Greve

Symbol CE:

Temperature limits

Use by

Please note the instructions for use

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